

# CHAMPIONS IMPLANT PASSPORT

Dear patient,

You were successfully provided with one or more dental implants. For documentation, the dental office freely provides you with the implant passport with all data concerning the placed implants (date, implant site, manufacturer, lot, diameter, and length). This a voluntary, free service by your dental office.

**Please keep this passport as it provides information for your dentist about your implantations even for decades.**

## Patient

Name

Street, Number

ZIP code, city

Date of birth

Dentist / Dental Office

After implantation, please follow the instructions listed below:

- 1) At home, cool the surgical site with cold pads, compresses, or wash gloves (do not cool with ice!).
- 2) Take painkillers and/or concomitant medication (antibiotics) as recommended by your Dentist.
- 3) The minimally invasive implantation remains a surgical intervention. During the first days after the intervention, you should avoid the sauna, the swimming pool, sports, and further physical efforts. In addition, do not smoke during that time. Treat the surgical site with care.
- 4) An inflammatory reaction reaches its peak only 48 hours after surgery.
- 5) During the first two months after implantation, you should not bite into food that is hard to chew and that causes excessive masticatory forces. If you wear a temporary denture, you should only eat soft food that is easy to chew.
- 6) In case of complications, consult the dentist immediately.
- 7) Brush your teeth and surgical site with a soft toothbrush and toothpaste at least 2 to 3 times a day for two weeks. After brushing your teeth, rinse your mouth with a mouthwash (for example with the Champions alfa Mouthwash).
- 8) Regular professional dental cleaning is recommended to ensure a successful dental implant treatment. Visit your dentist regularly.

We wish you a speedy recovery, and we hope that you will be pleased with your implant-supported dentures!

For further information and advice, please visit the homepage for patients at:

**champions-4-champions.com**

ImplantpassportA4EN-2022

<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>
<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>
<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>	<div>Date:</div> <div>Position:</div> <div>Further details:</div> <div>Label</div>